

# Broad spectrum cleaner and disinfectant

#### Description

Diversey Oxivir Plus Spray is a ready to use cleaner disinfectant for use on hard, nonporous surfaces, some soft furnishings and noninvasive medical devices.

#### **Key properties**

- Bactericidal, Virucidal and Yeasticidal.
- Patented Hydrogen Peroxide based acidic formulation
- Short contact times
- Effective Cleaner
- Fragrance free

#### **Benefits**

- Extensively tested against European norms and effective against multiple pathogens of concern, including HIV, HBV, HCV, Avian Influenza, Pseudomonas aeruginosa , Staphylococcus aureus, Salmonella enterica, MRSA, VRE and Norovirus
- Cleans and disinfects in one step, reducing working time
- Excellent cleaning performance, including descaling and lime limitation removes the need for a pre-cleaning step
- Good user safety profile, with no hazard warnings on RTU format; does not contain respiratory irritants
- AHP formulation is gentle on surfaces extending equipment life
- Good environmental profile. Active breaks down into oxygen and water
- Suitable for use in high risk areas in Healthcare, including OR, ICU, NICU and patient rooms

# **Use instructions**

#### Spray cleaning and disinfection:

- 1. Remove gross soils.
- 2. Apply to surfaces by either spraying the wiping cloth and wiping surfaces or applying directly to the surface through use of a trigger sprayer.
- 3. After the contact time, either wipe dry or allow to air dry.

# Dosage:

Ready to use, no dilution required.







A.I.S.E



# Technical data

Appearance: Clear, yellow liquid pH value (in use): 1.8 Relative density (20°C): 1.000 The above data is typical of normal production and should not be taken as a specification.

# Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet; sds.diversey.com. Store in original closed containers away from extremes of temperature. Only for professional users / specialists. Reacts with oxidisable substances.

# **Product compatibility**

Do not mix with other products. Do not use on acid-sensitive surfaces. Under recommended conditions of use, Diversey Oxivir Plus Spray is suitable for use on most materials commonly encountered.

### Microbiological data

Efficacy proven by compliance with Efficacy proven by compliance with: EN1276, EN13697, EN1650, EN14776, EN13624 and EN13727.

#### **Environmental information**

The surfactants used in this product are biodegradable in compliance with the requirement of EU Detergent Regulation, EC 648/2004.

#### Available pack sizes

Diversey Oxivir Plus Spray is available in 6x0.75L.